**EN** BlancOne® soft silicone Cheek Retractor & Lips Protection has a support structure that forces and keeps the jaw open, protecting Lips, during oral surgery and teeth whitening procedures. Place it between the patient's teeth to keep the mouth open. This retractor is reusable and can be autoclaved. Available in SMALL, MEDIUM and LARGE sizes.

#### Cautions:

1. Please check the retractor for damage or aging before each use.
2. Autoclave before each use.

#### Sterilization method:

1. Brush the dirt on the mouth opener with a soft brush.
2. Rinse the mouth opener with distilled water.
3. Remove any liquid residues (use a water-absorbent cloth and blow dry with compressed air).
4. Put the retractor in a vacuum autoclave at least 15 minutes at a temperature of 135°C±2°C and a pressure of 220KPa.

**5. Do not cold sterilize or immerse in chemicals, they can cause soft tissue burns.**

#### Storage conditions

Storage at normal temperature and humidity. This product should be placed or stored in a clean, dry environment away from harmful.

chemicals and gases such as acids and alkali.

#### Environmental protection:

This mouth opener does not contain harmful components and can be disposed of and destroyed according to local regulations.

#### MD

It's necessary to report any serious incident occurring in relation to the medical device supplied by us to the manufacturer and to the competent authority of the Member State in which you are based.

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