Quick post hygiene treatment	GB	2
Trattamento rapido post igiene orale	IT	4
Quick-Behandlung nach der Mundhygiene	DE	6
Traitement rapide post-hygiène dentaire	FR	8
Tratamiento rápido post higiene bucal	ES	10
Tratamento rápido pós-higiene oral	PT	12
Snelle behandeling na de mondhygiëne	NL	14
Hurtig tandblegning efter tandrensning	DA	16
Nopea hampaidenvalkaisu hampaiden puhdistuksen jälkeen	FI	18
Rask tannbleking etter tannrens	NO	20
Snabb klinikblekning efter tandrengöring	SV	22
Szybki zabieg przeprowadzany po higienie jamy ustnej	PL	24
Rychlé ošetření ihned po dentální hygieně	CZ	26
Rýchle ošetrenie zubov po ústnej hygieny	SK	28
Gyors poszt-higiéniás kezelés	HU	30
Tratament rapid post igienă orală	RO	32
Быстрое отбеливание после полости рта	RU	34
Ağız hijyeni sonrası hızlı tedavi	TR	36
معالجة سريعة بعد تنظيف الأسنان.	AR	38

Quick post hygiene treatment

BlancOne® CLICK is a quick post dental hygiene whitening treatment. This treatment offers a very popular upgrade to the dental hygiene session at a very low cost and in just 10 minutes.

BlancOne® CLICK has been designed to promote a complete whitening treatment (Touch or Ultra) and as a promotional and marketing tool for the office.

BlancOne® Whitening System

The BlancOne® ARCUS lamp and BlancOne photochemical whitening Gels (ULTRA, TOUCH, CLICK) are designed to work seamlessly together, playing a crucial role in ensuring user safety and the success of the whitening process. The ARCUS lamp is intended exclusively for BlancOne® photochemical whitening gels and is not to be used alone. Similarly, these gels are specifically formulated to be activated with the BlancOne® ARCUS lamp.

The BlancOne® photochemical gels (ULTRA, TOUCH, CLICK) require activation with the BlancOne ARCUS lamp to maintain user safety and optimize results. The distinct features of the ARCUS lamp, such as its wavelength and intensity, are vital for effectively activating these gels. Users should be protected with BlancOne® CARE gear for the eyes, face, and lips.

Dental professionals must receive training and certification to ensure they understand and are committed to following safety protocols with the BlancOne® In-Office Whitening System. The authorized dealer is charged with providing or facilitating this training and certification, ensuring that all system operators meet local standards of expertise and compliance.

Should any significant adverse reactions occur with the BlancOne® Whitening System, these should be reported to the authorized dealer. The dealer is then responsible for documenting the incident comprehensively and reporting it to the appropriate national health authorities and IDS Spa. However, common effects like mild and temporary sensitivity or gum irritation do not require reporting.

USE INSTRUCTIONS

A) Preparing the gel

1. Remove the caps of the two syringes (whitening gel and activating gel) and connect them together using the luer-lock connector.
2. Gently push the piston of one syringe to completely extrude the gel into the other syringe.
3. Now push the piston of the syringe with the gel, transferring it into the empty syringe.
4. Repeat the operation described in the previous step until the two gels (whitening and activator) are completely mixed, creating a single homogeneous gel.
5. Remove the connector from the syringe with the gel and attach the applicator tip.

B) Preparing the patient

1. The patient should be fitted with a BlancOne® gag with tongue depressor and then insert cylindrical tampons above and below the lips to isolate them from the area to be treated.
2. Remove the salivary film with a brush; if necessary, use a mix of water and pumice stone. DO NOT use polishing pastes or products containing fluorine. Rinse and dry.
3. Measure the patient's initial shade of white using the shade guide.

C) Whitening treatment

1. Gently push the piston of the syringe containing the whitening gel to apply the gel on the surface of all teeth to be bleached.
2. Photoactivate the whitening gel with a blue light.
3. Remove the excess whitening gel using a cylindrical cotton roll and suction away the gel residues, rinsing with the air/water syringe.

WARNING: The activated gel will be rather dense. If the suction spout becomes clogged, suction off with copious amounts of water.

NOTE:

the gingival edge might be slightly red in some points but this is a temporary effect that will disappear in about one hour.

D) End of treatment

1. Use the shade guide to measure the final shade of white of the patient's teeth.
2. Remove the gag and rinse the patient's mouth.

Photo activation of the gel

BlancOne® whitening gels require a suitable light source for the photochemical activation.

To guarantee the ideal whitening result, we recommend using BlancOne® ARCUS whitening lamp, which, being equipped with 10 powerful 5W led lights, guarantees the full activation of the gel simultaneously on both arches, including the sides, within the short time expected.

The head of BlancOne® ARCUS lamp should be positioned perpendicularly to the teeth at a distance of 5-10 cm from the mouth to guarantee a good ventilation.

Using other whitening lamps might reduce the effectiveness of the treatment, require a longer time or, in some cases, completely fail to activate the whitening gel.

RECOMMENDED ACTIVATION TIME

Equipment	Power	Type of activation	Time
BlancOne® ARCUS Lamp	250 mW/cm ²	simultaneous on both arches	10 minutes

EUROPEAN UNION WARNINGS

- 5,6% of H₂O₂ is released.
- Not to be used on anyone under 18 years of age.
- To be sold only to dental practitioners.
- To be used only by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Only then can it be made available to the patient, to complete the course of treatment.

PRECAUTIONS

- A dental examination is mandatory prior to bleaching.
- Bleaching will not lighten porcelain crowns and dental restorations.
- For at least 6 hours after the bleaching procedure do not use tobacco products and do not drink coffee or red wine.
- Not intended for use by pregnant women. Consult a doctor if you are breast feeding.
- Do not swallow the gel. Avoid contact with tissues other than teeth.
- Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
- Gum tissue irritation may result if used in individuals with periodontal disease.
- Keep teeth bleaching products away from children.
- Avoid contact with skin, mucous membranes and eyes. In case of contact with EYES: rinse eyes carefully with water for several minutes and request medical advice/attention. ON SKIN/ MUCOUS MEMBRANES: wash with copious amounts of water. If irritation occurs request medical advice/ attention.
- If swallowed in small quantities the product is harmless, otherwise consult a doctor.
- Some users may experience sensitivity or gingival irritation. While these are mild and transient in most cases, sometimes treatment by a dentist may be required.

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