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Instructions for the patients

BlancOne® HOME is a cosmetic whitening treatment to be used at home with individual trays, under the supervision of a dentist. BlancOne® HOME is available in two versions: NIGHT and DAY according to the patient's preference for a treatment performed during the day or the night.

BlancOne® HOME low viscosity composition prevents dehydration, considerably reducing sensitivity, its physiological pH is not aggressive towards the enamel guaranteeing the highest effectiveness of the gel.

BlancOne® HOME contains hydroxyapatite, strengthening the enamel and hyaluronic acid to protect gums.

Preparation and storage of the whitening gel

BlancOne® HOME whitening gel is a two-component material to be prepared before the use by mixing the two components.

- Remove the cap from the syringe with the whitening gel (Whitening) and insert the connector, included in the package, turning clockwise to block it.
- Remove the cap from the syringe with the activator gel (Activator) and connect the syringe to the other side of the connector firmly, turning the syringe clockwise to block it.
- Pushing the piston of one of the two syringes, transfer the content of a syringe into the other.
- After, push the piston of the other syringe to completely transfer the content to the empty one.
- Repeat the last passage 10-15 times to obtain the complete mixing of both gels.
- Completely transfer the gel in a syringe and remove the connector and the empty syringe.
- Connect the application tip to the filled syringe, detach the tip end-cap and close well the syringe turning from the other side the tip end-cap and placing this at the opening of the tip.
- Store the syringe in the fridge for further applications and up to 10 days.

Whitening gel application

The treatment consists in the daily application of the gel by means of the

personalised trays.

The length of the treatment can vary on the basis of the clinical needs, treatment goals and possible combination with in-office treatments. Do not exceed the number of applications suggested by the dentist.

Interrupting the treatment for a few days does not compromise the whitening effect once the treatment is resumed.

- Remove the syringe from the fridge and leave it at room temperature for 30 minutes before using it.
- Apply the gel in the front (vestibular) part of the trays at about half height from molar to molar.
- Dry the teeth surface with a cloth.
- Wear the trays once filled, remove the gel in excess and leave on for 2-3 hours in case of daily treatment (DAY) or overnight for the night treatment (NIGHT).
- Remove the trays and clean both teeth and trays from any residual of gel using brush and water.
- Dry and store the trays in the specific tray holder box.
- Do not eat or drink for at least 30 minutes.

European Union warnings

- The final product, after mixing the two components, contains:
 - NIGHT 3.5% hydrogen peroxide as 10% carbamide peroxide
 - DAY 5.6% hydrogen peroxide as 16% carbamide peroxide
- Do not use on patients under 18 years of age.
- It can be solely sold to dental practitioners.

- For each treatment cycle, the first use must be carried out in office or under the supervision of a dentist, where the same level of safety is guaranteed. Only after this it can be made available to the patient to complete the treatment cycle.
- It contains sodium fluoride (500 ppm).

Precautions

- Keep away from children.
- It contains peroxide and fluoride, in case of ingestion in large quantities it can be dangerous. In this case consult a doctor.
- Avoid contact with the eyes. In case of contact immediately rinse with abundant water.
- Not suitable for pregnant women. If breastfeeding, consult a doctor.
- Some patients can feel sensitivity or gingival irritation. These are generally mild and temporary effects. If the inconvenience persists, interrupt the treatment and make an appointment to assess the problem.
- Food and fruit juices can cause tooth sensitivity.
- Do not smoke or chew tobacco or eat during the whitening procedure
- The treatment does not whiten crowns, inlays, fillings, non-vital teeth.
- Once mixed, store in the fridge. Do not freeze.
- Dispose of the unused gel after having completed the treatment.
- In case of doubts on the correct use of this product consult the dentist as soon as possible.

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