Protective kit for the patient

BlancOne[®] SAFETY KIT is part of a line of BlancOne[®] accessories for the protection and comfort of the patient during in office whitening treatments with an arc light.

GB BlancOne[®] SAFETY KIT includes facial protections and gauze to protect the tissues both from the heat produced by the whitening lamp and from accidental burns resulting from the use of hydrogen peroxide in high percentages (BlancOne[®] ULTRA).

BlancOne[®] ARCUS

If used according to BlancOne[®] protocols, the whitening lamp BlancOne[®] ARCUS is safe for teeth and gums. However, we know that whitening lamps produce on the skin a heat similar to that of a tanning lamp. It's therefore **important to maintain a working distance of 5-10 cm between the lamp head and the patient's mouth**, this will allow air to circulate avoiding thus overheating which will result in a burning feeling on the patient's lips and face.

INSTRUCTIONS FOR USE (see images)

1. Insert the BlancOne[®] CHEEK RETRACTOR with tongue guard.
2. Fix the cheek retractor loops to the facial protection.

3. In case of BlancOne[®] ULTRA treatments, More protective gauze can be positioned (folded) instead of dental rolls to distance and insulate lips and cheeks from the application area of the whitening gel.
4. Cover the lower lip with a protective gauze. In the BlancOne[®] ULTRA treatments, it's necessary to apply BlancOne[®] GINGIVAL BARRIER along the gingival margin as per instructions for use.

Before activating the lamp BlancOne[®] ARCUS it's necessary to wear and have the patient wear protective goggles.

BlancOne[®] SAFETY KIT includes also Vitamin E Swabs to resolve soft tissue bleaching or irritation. Instructions:

- Hold the vitamin E swab vertical with the red line at the top
- Break along the red line and allow the vitamin E to slowly drain to the bottom of the swab
- Swab the lower saturated cotton portion on gums, lips, hands etc. as necessary.

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