BLANCONE SEAL Sealer after bleaching

EN

Instructions for use

Product description

Smoking and drinking coffee or tea are habits that can quickly compromise the result of a teeth bleaching procedure. It is usually recommended to stop smoking and drinking coffee and tea for at least 72 hours after bleaching, when teeth are more likely to absorb new discolorations. Nevertheless it is a very difficult for patients to comply with such a limit.

BlancOne® SEAL is a silicone based, acid protecting varnish for sealing teeth after bleaching procedures. It forms a film that protects teeth from new discolorations from acidic food, coloring beverages and smoke, thus limiting new discolorations and making teeth bleaching more durable.

BlancOne® SEAL varnish contains Nano-Fluorapatite and Nano-Calciumfluoride. Nano-technology fluorapatite particles are known to enhance remineralization of tooth material. BlancOne® SEAL contains 2000 ppm (0.2 %) fluoride. BlancOne® SEAL is a one component and does not require etching. It builds a protective film, that can be renewed or repaired just as you like, if necessary. The application is easy and time saving. One kit is usually enough for approximately 20 patients.

The intense smell of BlancOne® SEAL volatilizes during drying within 1 minute.

BlancOne® SEAL contains ethylacetate. Ethylacetate is highly flammable. Keep away from sources of ignition. Avoid inhalation of ethylacetate vapour.

After eye contact rinse out with plenty of water and call an ophthalmologist. After contact with skin, wash with plenty of water and soap.

Avoid contamination of the BlancOne® SEAL with dirty disposable brushes.

Keep away from children!

Indications/Intended use

Sealing teeth after bleaching procedures

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Impressions with vinyl silicones can be made earliest 4 weeks after the treatment with BlancOne® SEAL.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application of BlancOne® SEAL after teeth bleaching

1. Before application vigorous shake the bottle of BlancOne® SEAL.
2. During shaking listen to the glass bead inside the bottle.
3. Leave lip retractor in place to make application easier.
4. Remove gum protection, in case it was used during bleaching treatment.
5. Use disposable cups and applicators provided in the package, one dip is enough to cover approximately 3 teeth.
6. Close the bottle immediately after use.
7. Apply BlancOne® SEAL 2-3 times on each tooth in an even thin layer.
- If teeth have been bleached on the back, apply the varnish there as well.
- BlancOne® SEAL may advantageously be applied to tooth areas exposed by receding gums and in between teeth.

Limitation of Liability

Except where prohibited by law, SanaPro Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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PT

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Product description

Let it dry for 1 minute or dry it gently in an oil free air stream.

8. Let it dry for 1 minute or dry it gentle in an oil free air stream.

9. Thoroughly rinse the patient's teeth after completing sealing.

Important remarks

In the first 2 hours after application of BlancOne® SEAL don't eat anything nor brush the teeth.

• Do not seal teeth if bleaching treatment is not completed. In case of follow-up with take home bleaching treatments, perform sealing after completion of home treatment.

• Air abrasion treatment is recommended before bleaching teeth that have been previously sealed.

• The intense smell of BlancOne® SEAL volatilizes during drying within 1 minute.

• BlancOne® SEAL contains 2000 ppm (0.2 %) fluoride.

BlancOne® SEAL is a one component and does not require etching. It builds a protective film, that can be renewed or repaired just as you like, if necessary. The application is easy and time saving. One kit is usually enough for approximately 20 patients.

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• BlancOne® SEAL contains ethylacetate.

Ethylacetate is highly flammable.

Keep away from sources of ignition. Avoid inhalation of ethylacetate vapour.

After eye contact rinse out with plenty of water and call an ophthalmologist.

After contact with skin, wash with plenty of water and soap.

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