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## Teeth whitening pen

- GB** *BlancOne® STICK is a home teeth whitening treatment. Used for two weeks, it will make your smile brighter and whiter. Ideal for removing stains from tobacco, coffee, tea as well as pigmented foods and beverages. Suitable for long-term maintenance of in-office professional teeth whitening results.*

### USE INSTRUCTIONS

- 1) Clean your teeth by brushing.
- 2) Remove the cap and turn the bottom of the pen (3-4 clicks) clockwise to wet the brush with a quantity of gel. (IMPORTANT NOTE: during the first use more clicks may be required in order to break the safety seal).
- 3) Apply the gel on the teeth and rub thoroughly on their surface for 2 minutes. During this period, regularly rewet the brush with a new quantity of gel, turning the bottom clockwise and reapply. Usually 10-12 clicks are needed to apply an adequate amount of whitening gel per procedure.
- 4) Spit out the remaining gel without rinsing with water and do not eat or drink anything for 30 minutes.
- 5) After the end of the procedure clean the silicon brush with a tissue and recap the pen.

### RECOMMENDATIONS

**Stabilisation** – To be used once a day, for 2 weeks, immediately after the in-office treatment to stabilise results

**Maintenance** – A one-week cycle can be performed, maximum once a month, by using the pen once a day to refresh the teeth shade of white.

**Stain removal** – To be applied in the morning and in the evening, brushing the stained surface thoroughly. To be repeated until the stain is removed.

**Make-up** – On special occasions the gel can be applied on both arches to brighten your smile.

### EUROPEAN UNION WARNINGS

- 5,6% of H<sub>2</sub>O<sub>2</sub> is released.
- Not to be used on anyone under 18 years of age.
- To be sold only to dental practitioners. For each use cycle, the first application must be done only by dental practitioners or under their direct supervision if an

equivalent level of safety is ensured. Afterwards the product will be provided to the consumer to complete the use cycle.

### PRECAUTIONS

- A dental examination is mandatory prior to bleaching.
- Bleaching will not lighten porcelain crowns and dental restorations.
- For at least 6 hours after the bleaching procedure do not use tobacco products and do not drink coffee or red wine.
- Not intended for use by pregnant women. Consult a doctor if you are breast feeding.
- Avoid contact with tissues other than teeth.
- Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
- Gum tissue irritation may result if used in individuals with periodontal disease.
- If any irritation of gums or tooth sensitivity occurs, discontinue the use of the product and consult your dentist.
- Keep the product out of children's reach.
- Avoid contact with the eyes. In case of contact rinse eyes carefully with water for several minutes and request medical advice/attention.
- Do not apply quantities greater than those required for the whitening process. If swallowed in small quantities the product is harmless, otherwise consult a doctor.
- Some users may experience sensitivity or gingival irritation. While these are mild and transient in most cases, sometimes treatment by a dentist may be required.
- Do not use longer than 14 consecutive days; a 2-week stop is recommended between each application cycle (14 days).

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