In-Office Cosmetic Teeth Whitening	EN
Sbiancamento dentale cosmetico in studio	IT
Kosmetische Zahnaufhellung in der Praxis	DE
Blanchiment cosmétique des dents au cabinet	FR
Blanqueamiento dental cosmético en la clínica	ES
Branqueamento dentário cosmético no consultório	PT
Cosmetisch tanden bleken in de praktijk	NL
Kosmetisk tandblegning på tandklinikken	DA
Hampaiden kosmeettinen valkaisu vastaanotolla	FI
Kosmetisk tannbleking hos tannlegen	NO
Kosmetisk tandblekning på klinik	SV
Gabinetowe kosmetyczne wybielanie zębów	PL
Kosmetické bělení zubů v ordinaci	CZ
Ambulantné kozmetické bielenie zubov	SK
Rendelői kozmetikai fogfehérítés	HU
Albirea cosmetică a dinților în cabinet	RO
Косметическое отбеливание зубов в кабинете	RU
Muayenehanede Kozmetik Diş Beyazlatma	TR
تبييض الأسنان التجميلي داخل العيادة	AR

www.blancone.eu/pro

IDS Spa
17100 Savona - Italy - Via San Cristoforo, 28/10
Tel. +39 019 862080 - www.idsdental.it

Rev.1 Feb, 2025

BlancOne® TOUCH+ is an advanced cosmetic teeth whitening gel containing 6% hydrogen peroxide and photo accelerators. The light activates it quickly, offering a modern solution specifically for cosmetic enhancement. This system can significantly boost tooth whiteness in 30 minutes without protective barriers while ensuring minimal sensitivity.

BlancOne® TOUCH+ blends the whitening capabilities of peroxides with a natural remineralization process through hydroxyapatite (HAP), effectively reducing sensitivity. The gel is prepared using a self-mixing tip that combines a whitening gel with an activator/remineralizing gel.

BlancOne® Whitening System

The BlancOne® ARCUS lamp and BlancOne® photochemical whitening Gels (ULTRA, TOUCH, CLICK) are designed to work seamlessly together, playing a crucial role in ensuring user safety and the success of the whitening process. The ARCUS lamp is intended exclusively for BlancOne® photochemical whitening gels and is not to be used alone. Similarly, these gels are specifically formulated to be activated with the BlancOne® ARCUS lamp.

The BlancOne® photochemical gels (ULTRA, TOUCH, CLICK) require activation with the BlancOne® ARCUS lamp to maintain user safety and optimize results. The distinct features of the ARCUS lamp, such as its wavelength and intensity, are vital for effectively activating these gels. Users should be protected with BlancOne® CARE gear for the eyes, face, and lips.

Dental professionals must receive training and certification to ensure they understand and are committed to following safety protocols with the BlancOne® In-Office Whitening System. The authorized dealer is charged with providing or facilitating this training and certification, ensuring that all system operators meet local standards of expertise and compliance.

Should any significant adverse reactions occur with the BlancOne® Whitening System, these should be reported to the authorized dealer. The dealer is then responsible for documenting the incident comprehensively and reporting it to the appropriate national health authorities and IDS Spa. However, common effects like mild and temporary sensitivity or gum irritation do not require reporting.

USAGE GUIDELINES

A) Recipient Preparation

1. Before initiating the whitening session, conduct a thorough examination to assess the health of oral tissues and address any dental issues or decay.
2. Allow any damaged or recovering oral tissues time to heal before the whitening process.
3. Clearly explain the procedure to the individual

receiving the service and secure their informed consent beforehand.

4. Where necessary, remove tartar and surface stains in a professional dental cleaning. Use a brush or powders for this purpose, avoiding polishing pastes. Ensure the area is rinsed and completely dry afterward.
5. Equip the individual with a BlancOne® cheek retractor with a tongue depressor, facial protection, and protective goggles. A gingival barrier is not necessary unless there is gum irritation.
6. Use a shade guide to record the initial color of the teeth in the individual's record or digitally with the BlancOne® PRO app. Also, take a photo before starting, under neutral lighting, for comparison.

B) Gel Preparation and Storage

1. Remove the cap from the syringe by twisting it counterclockwise.
2. Attach the automix tip, securing it by twisting it clockwise.
3. After use, replace the cap on the syringe for storage.

C) Teeth Whitening Process

1. Gently press the syringe's piston to distribute the whitening gel across the teeth intended for the procedure, aiming for uniform coverage approximately 1-2mm thick.
2. Use the ARCUS activating light to activate the whitening gel.
3. Use a cylindrical cotton roll to remove any excess gel, and clear away any remaining gel with suction – do not rinse.
4. BlancOne® TOUCH+ involves three whitening cycles in a single session. Perform the whitening steps (C 1-3) two additional times. If desired, a fourth cycle may be conducted for more pronounced results.

Note: Once activated, the gel will become thick in texture. If the suction tool gets clogged, rinse it well with plenty of water.

Photo Activation of the Gel - Whitening Protocol for Vital Teeth

Position the head of the BlancOne® ARCUS lamp so it's perpendicular to the teeth, keeping it 5-10 centimeters (2-4 inches) away from the mouth to promote adequate airflow. Recommended Light Intensity and Activation Times:

- **Fast:** High Intensity for 8 minutes for a rapid activation.
- **Standard:** High Intensity for 10 minutes for efficient gel activation.
- **Extended:** High Intensity for 12 minutes, ideal for those without sensitivity, to enhance gel activation.
- **Sensitive:** Low Intensity for 10 minutes, suitable for individuals with sensitive teeth or sensitivity issues.

D) Finishing the Process

1. Remove any gingival barrier and rinse the tooth surfaces using the air/water syringe, clearing away any leftover gel.
2. Use the shade guide to evaluate the final shade of white achieved and take a photo afterward in neutral lighting for comparison.
3. Please remove the cheek retractor and ask the individual to rinse their mouth thoroughly.
4. Present the whitening results to the individual with a mirror and the photos taken.
5. Offer guidance on post-session care and recommend any necessary follow-up steps.

Warnings

- The mixed gel contains peroxides (6.0% H₂O₂)
- To be only sold to dental practitioners.
- For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Not advised for pregnant individuals.
- Consult a healthcare provider if breastfeeding and considering oral care procedures.
- Not to be used on a person under 18 years of age.
- Ensure that the user does not have allergies to peroxides or any ingredients in the product before use.
- Exercise caution for individuals with light sensitivity.
- A dental professional must evaluate oral health, including teeth and gums, before beginning the whitening process. It is not recommended for those with gum disease.
- Protective gloves and eyewear should be worn during application.
- Avoid swallowing the gel and contact with soft oral tissues.
- Swallowing small amounts is not considered harmful, but rinse with water or seek medical advice if it occurs.
- Avoid contact with eyes. Rinse immediately if product comes into contact with them.
- If the gel contacts skin or mucous membranes, rinse well with water and apply vitamin E or hyaluronic acid gel if irritation develops.
- Store out of reach of children.
- Temporary mild sensitivity or irritation of the gums may occur for some users. Cease use and consult a dental professional if discomfort continues.
- Whitening procedures do not change the color of porcelain crowns or dental restorations.
- Avoid tobacco, tea, coffee, or red wine for at least 3 hours after the session.
- Sensitivity to hot, cold, or acidic foods and drinks may increase after the whitening session.

MADE IN GREECE
FOR PROFESSIONAL USE

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