In office single session treatment	GB	2
Trattamento in studio mono seduta	IT	4
Einmalbehandlung in der Praxis	DE	6
Traitemet au fauteuil en une séance	FR	8
Tratamiento en consultorio en una sola sesión	ES	10
Tratamento de sessão única em consultório	PT	12
Eenmalige behandeling in de praktijk	NL	14
Blegningsbehandling i en enkelt session på tandklinik	DA	16
Vastaanottovalkaisu yhdellä käyntikerralla	FI	18
Blekningsbehandling i én enkelt økt på tannklinik	NO	20
Blekningsbehandling i en enda session på tandklinik	SV	22
Zabieg wykonywany w gabinecie w ramach jednej wizyty	PL	24
Ošetření během jedné návštěvy v zubní ordinaci	CZ	26
Ambulantré bielenie zubov počas jednej návštevy	SK	28
Egy alkalmas rendelői fogfehérítő kezelés	HU	30
Tratament în cabinet, într-o singură sedință	RO	32
Процедура в стоматологическом кабинете за один сеанс	RU	34
Tek seanstta klinik tedavisi	TR	36
معاملة تبييض الأسنان في جلسة واحدة بالعيادة	AR	38

www.blancone.eu/pro

In office single session treatment

BlancOne[®] TOUCH is an in-office treatment that in just 30 minutes will immediately and effectively whiten teeth without the use of protective dams and without any dentin sensitivity.

USE INSTRUCTIONS

A) Preparing the gel

1. Remove the caps of the two syringes (whitening gel and activating gel) and connect them together using the luer-lock connector.
2. Gently push the piston of one syringe to completely extrude the gel into the other syringe.
3. Now push the piston of the syringe with the gel, transferring it into the empty syringe.
4. Repeat the operation described in the previous step until the two gels (whitening and activator) are completely mixed, creating a single homogeneous gel.
5. Remove the connector from the syringe with the gel and attach the applicator tip.


B) Preparing the patient

1. The patient should be fitted with a **BlancOne[®]** gag with tongue depressor and then insert cylindrical tampons above and below the lips to isolate them from the area to be treated.
2. Remove the salivary film with a brush; if necessary, use a mix of water and pumice stone. DO NOT use polishing pastes or products containing fluorine. Rinse and dry.
3. Measure the patient's initial shade of white using the shade guide.

C) Whitening treatment

1. Gently push the piston of the syringe containing the whitening gel to apply the gel on the surface of all teeth to be bleached.
2. Cure the whitening gel with a blue light.

Once activated, the gel will be swollen and full of oxygen (bubbles) as shown in the figure.

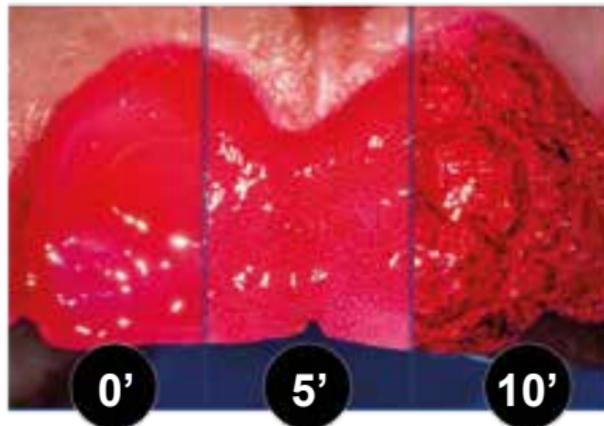3. Remove the excess whitening gel using a cylindrical cotton roll and suction away the gel residues, rinsing with the air/water syringe.
WARNING: The activated gel will be rather dense.
If the suction spout becomes clogged, suction off with copious amounts of water.
4. The **BlancOne[®] TOUCH** treatment consists of 3 whitening cycles in a single session.
Repeat the whitening cycle (C 1-3) 2 more times.
A fourth cycle can be added to further stabilize the result.

D) End of treatment

1. Remove any gingival barrier. Use the air/water syringe to rinse the surface of the teeth, suctioning off any gel residues.
NOTE: the gingival edge might be slightly red in some points but this is a temporary effect that will disappear in about one hour.
2. Use the shade guide to measure the final shade of white of the patient's teeth.
3. Remove the gag and rinse the patient's mouth.

Photo activation of the gel

BlancOne® whitening gels require a suitable light source for the photochemical activation.

To guarantee the ideal whitening result, we recommend using BlancOne® ARCUS whitening lamp, which, being equipped with 10 powerful 5W led lights, guarantees the full activation of the gel simultaneously on both arches, including the sides, within the short time expected.

The head of BlancOne® ARCUS lamp should be positioned perpendicularly to the teeth at a distance of 5-10 cm from the mouth to guarantee a good ventilation.

Using other whitening lamps might reduce the effectiveness of the treatment, require a longer time or, in some cases, completely fail to activate the whitening gel.

RECOMMENDED ACTIVATION TIME

Equipment	Power	Type of activation	Time
BlancOne® ARCUS Lamp	250 mW/cm ²	simultaneous on both arches	8 minutes

EUROPEAN UNION WARNINGS

- 5,6% of H₂O₂ is released.
- Not to be used on anyone under 18 years of age.
- To be sold only to dental practitioners.
- To be used only by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Only then can it be made available to the patient, to complete the course of treatment.

PRECAUTIONS

- A dental examination is mandatory prior to bleaching.
- Bleaching will not lighten porcelain crowns and dental restorations.
- For at least 6 hours after the bleaching procedure do not use tobacco products and do not drink coffee or red wine.
- Not intended for use by pregnant women. Consult a doctor if you are breast feeding.
- Do not swallow the gel. Avoid contact with tissues other than teeth.
- Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
- Gum tissue irritation may result if used in individuals with periodontal disease.
- Keep teeth bleaching products away from children.
- Avoid contact with skin, mucous membranes and eyes. In case of contact with EYES: rinse eyes carefully with water for several minutes and request medical advice/attention.

ON SKIN/ MUCOUS MEMBRANES: wash with copious amounts of water. If irritation occurs request medical advice/ attention.

- If swallowed in small quantities the product is harmless, otherwise consult a doctor.
- Some users may experience sensitivity or gingival irritation.

While these are mild and transient in most cases, sometimes treatment by a dentist may be required.

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