# BLANCONE®

ULTRA<sup>+</sup>

Tooth discoloration treatment	EN	2
Trattamento delle discromie dentali	IT	5
Behandlung von Zahnverfärbungen	DE	8
Traitement des dyschromies dentaires	FR	11
Tratamiento de las discromías dentales	ES	14
Tratamento das discromías dentárias	PT	17
Behandeling van intense tandverkleuring	NL	20
Behandling af misfarvning af tænder	DA	23
Hampaiden värjäytymien käsittelyhoito	FI	26
Behandling av misfärging av tennene	NO	29
Behandling av missfärgade tänder	SV	32
Uzuwanie przebarwień zębów	PL	35
Ošetření diskolorovaných zubů	CZ	38
Ošetrenie diskolorácie zubov	SK	41
Tratament pentru albirea dentară	RO	44
Система для клинического отбеливания зубов	RU	47
Diş Beyazlatma Tedavisi	TR	50
علاج تصبغ الاسنان	AR	53

[www.blancone.eu/pro](http://www.blancone.eu/pro)

## Tooth discoloration treatment

### DESCRIPTION AND INDICATIONS

BlancOne<sup>®</sup> ULTRA<sup>+</sup> is a tooth whitening gel based on 35% hydrogen peroxide and photo accelerators achieving a fast photochemical activation when irradiated by a suitable source of light.

BlancOne<sup>®</sup> ULTRA<sup>+</sup> melds the whitening capabilities of peroxides with a natural remineralization process through hydroxyapatite (HAP), effectively minimizing sensitivity.

BlancOne<sup>®</sup> ULTRA<sup>+</sup> is ideal for an in-office treatment of the most problematic intrinsic tooth discolorations, like those caused by pharmacological therapies and whitening of non-vital teeth.

BlancOne<sup>®</sup> ULTRA<sup>+</sup> is a medical whitening treatment to be used solely by dental professionals in the dental field. Its use might be limited by local regulations in some EU countries.

### INSTRUCTIONS FOR USE

Before whitening, a thorough dental examination is necessary to verify the presence of possible problematic conditions or risks in performing the treatment.

#### A. Patient's preparation

At least one week prior to the treatment, patients should have gone through a professional dental hygiene session.

Explain the procedure to your patients and ask them to sign the informed consent before starting.

1. Ask your patients to wear the BlancOne<sup>®</sup> mouth gag with tongue guard and insert pads or gauzes under the lips and in gaps to separate the soft tissues from the treatment area.
2. Remove the salivary film with a brush or a prophylaxis cup. In case, use a mix of water and pumice. DO NOT use polishing paste or products containing fluoride. Rinse and dry.
3. Take the patient's initial shade of white using the colour guide, reorganised for whitening treatments, and take a pre-treatment picture, making sure the light is neutral.
4. Isolate gums by applying BlancOne<sup>®</sup> Gingival Barrier liquid dam and light cure following the instructions.
5. In case of patient's skin sensitivity to light or heat, protect face and lips with BlancOne Face Mask, protective gauzes and creams with a high protection against the sun.
6. Ask your patient to wear protective eyewear.

ADVICE: you can use the BlancOne<sup>®</sup> PRO app to record treatments and take pictures.

#### B. Preparation of the whitening gel

Each BlancOne Cycle Pack contains a whitening gel syringe (Whitening Gel), a syringe with photo accelerators (Photo Booster), a connector and tip for application.

1. Remove the caps from the two syringes and connect them by means of the connector.
2. Mix the two components by transferring them several times from one syringe to the other until you achieve a homogeneous mix (about 20 passages back and forth).
3. Transfer the full amount of the mixed gel in one of the two syringes.
4. Insert the transparent applicator tip.

The gel just mixed guarantees maximum effectiveness. The quantity of gel obtained with

each Cycle Pack is necessary and sufficient for an application.

**WARNING:** The gel is not sufficient for 2 applications, the layer would be too thin, furthermore, the residual gel will start its auto-activation losing effectiveness between one application and the next.

### C. Teeth Whitening Process

1. Gently press the syringe's piston to distribute the whitening gel across the teeth intended for the procedure, aiming for uniform coverage approximately 1-2mm thick.
2. Use the ARCUS activating light to activate the whitening gel.
3. Use a cylindrical cotton roll to remove any excess gel, and clear away any remaining gel with suction – do not rinse.
4. BlancOne® ULTRA+ involves three whitening cycles in a single session. Perform the whitening steps (C 1-3) two additional times. If desired, a fourth cycle may be conducted for more pronounced results.

**Note:** Once activated, the gel will become thick in texture. If the suction tool gets clogged, rinse it well with plenty of water.

### Photo Activation of the Gel - Whitening Protocol for Vital Teeth

Position the head of the BlancOne® ARCUS lamp so it's perpendicular to the teeth, keeping it 5-10 centimeters (2-4 inches) away from the mouth to promote adequate airflow. Recommended Light Intensity and Activation Times:

- **Fast:** High Intensity for 8 minutes for a rapid activation
- **Standard:** High Intensity for 10 minutes for efficient gel activation.
- **Extended:** High Intensity for 12 minutes, ideal for those without sensitivity, to enhance gel activation.
- **Sensitive:** Low Intensity for 10 minutes, suitable for individuals with sensitive teeth or sensitivity issues.

### D. Treatment completion

1. Remove the gingival barrier.
2. Rinse and polish the teeth surface with a brush or a prophylaxis cup.
3. Take the patient's final shade of white and a post-treatment picture, making sure the light is neutral.
4. Remove glasses, mouth gag and protections and ask the patient to rinse accurately.
5. Show your patient the result with a mirror and by means of the pictures.
6. Supply the patient with post-treatment instructions for a correct maintenance and follow-up.

**NOTE:** In-office BlancOne treatments are fast and gentle, normally they do not dehydrate the teeth and they don't create sensitivity, therefore following a particular diet or taking analgesics in advance is not necessary.

**WARNING:** In the presence of irritated or bleached gums, apply vitamin E.

### E. Endodontic treatment of non-vital teeth

1. Completely remove the filling material from the pulp chamber.
2. Prepare the pulp chamber by sealing with glass ionomer cement, approximately up to the bone edge, so that the gel might act in the visible part of the tooth crown.
3. Isolate the tooth from the mucous membranes by means of BlancOne Gingival Barrier liquid dam and light cure according to instructions.
4. Prepare the whitening gel as detailed in point B.
5. Apply the gel in the canal and on the vestibular surface.
6. Photo activate the gel both through the canal cavity and on the vestibular surface using the curing light (60 seconds).
7. Leave the photo activated gel to act for 3 minutes.
8. Suction off the gel, rinse and dry with air.

9. Repeat passages 5-8 another 2 times (3 in total).
10. Close and seal the access.

In case the result obtained with 3 passages is not satisfactory, proceed with a specific treatment for teeth treated endodontically with "walking bleaching" technique.

#### **Simultaneous treatment with vital teeth**

Whitening of endodontically treated teeth can also be carried out simultaneously with the treatment of vital teeth.

In this case, above detailed protocol will only involve the canal area, while the vestibular part of the tooth will be treated together with the other vital teeth.

Prior to each whitening cycle of vital teeth, as described in point C, follow passages 5 and 6 of the endodontic whitening protocol.

Leave the gel to act in the canal while the vestibular treatment is being performed, then suction off, rinse and dry.

Repeat the whole procedure another two times (3 in total), then close and seal the access.

#### **F. Warning**

Do not use on:

- Pregnant women
- Patients with periodontal conditions,
- Patients allergic to peroxides,
- Patients with hypersensitivity to light.

Seek medical advice if breastfeeding.

Wear protective gloves and eyewear.

Keep tooth whitening products away from children.

Do not swallow the gel. Avoid contact with tissues different from teeth.

In case of contact WITH EYES: thoroughly rinse eyes for several minutes and seek medical advice. Contact WITH SKIN/MUCOUS MEMBRANES: wash with abundant water. In case of irritation, apply vitamin E.

INGESTION: In case of accidental ingestion of small quantities, the product is not to be considered as dangerous, rinse with water. Alternatively seek medical advice.

Whitening has no effect on porcelain crowns and dental restorations.

Do not use products with tobacco; do not drink tea, coffee or red wine after the treatment for at least 3 hours.

Acid food/juices, both hot or cold can cause sensitivity if consumed just after whitening.

#### **MD**

It is necessary to report any serious incident occurring in relation to the medical device supplied by us to the manufacturer and to the competent authority of the Member State in which you are based.

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