BLANCONE®

ULTRA

Biophotonic Teeth Discoloration Treatment
Trattamento Biofotonico delle Discromie Dentali
Biophotoniche Behandlung zur Zahnentfärbung
Terapia Biofotónica de Decoloración Dental

INSTRUCTIONS FOR USE

INDICATIONS

Treatment of intrinsic tooth discoloration following pharmacological therapies.
For professional use only in the dental sector.

1. INTENDED USE

BLANCONE is a tooth discoloration treatment system intended only for in-office use by dental professionals and dentists. The kit is designed to be used with a dental curing light (as described below). This efficient and safe tooth discoloration treatment system eliminates intrinsic stains caused by certain drugs and restores the normal physiological color. BLANCONE works effectively on living and non-living teeth (treatment through the endodontic canal), but will not have any effect on porcelain teeth, resin prostheses, composites and amalgam.

2. KIT COMPOSITION

One kit contains:

- 1 5 ml ampoule of hydrogen peroxide 39% and stabilizer (final gel concentration: 29%).
- 1 dispenser syringe containing 1.6 grams (in powder) of BLANCONE biophotonic accelerator. The powder also contains thickeners, photochemical mediators, colorants and photoactivation indicators.
- 1 ready-to-use syringe containing 1.2 ml of methacrylic polymer protective gingival gel (BLANCONE protective gingival gel).
- 1 piston for the active gel dispenser syringe.
- 2 needles: a black one for the protective gingival gel syringe and a white one for the active gel syringe.
- 1 spatula to prepare the active gel by mixing the powder in the syringe cylinder with the hydrogen peroxide in the ampoule.

3. PRODUCT STORAGE

Store the product at a temperature between 5 °C and 25 °C in a cool, dry, and if possible, ventilated place. Keep away from heat and light.

4. EQUIPMENT NECESSARY FOR THE SYSTEM BUT NOT SUPPLIED

No purchase of a new special light is required.

To correctly and rapidly activate the gel, one of the following curing lights can be used:

- LED LIGHT
- HALOGEN LIGHT
- METAL HALIDE LIGHT
- PLASMA ARC LIGHT

Minimum light power required: 1000 mW/cm²

ACTIVATION TIME

The minimum photoactivation time for the active gel depends on the light power released by the light source: from approximately 5 seconds to 20 seconds per tooth. It should be recalled that a colour change indicates that the gel has been activated. As soon as the gel on one tooth is activated, move the light onto the next tooth. Exposing a tooth on which the gel has already been activated to extra light will not increase effectiveness and may cause patient discomfort.

BLANCONE ARCUS ARC LIGHT

We have developed BlancOne ARCUS, a powerful arc light, to simplify and speed up the entire procedure.

This light can photoactivate both dental arches simultaneously in just 3 minutes.

5. WARNINGS

WARNING

WARNING: the hydrogen peroxide contained in the kit is corrosive and comburent. ALWAYS WEAR PROTECTIVE EYEWEAR AND GLOVES.

COUNTER-INDICATIONS AND PRECAUTIONARY MEASURES

The product is not recommended:

- for use in patients under 15 years of age;
- for use on pregnant women or nursing mothers. In the event that a patient experiences sensitivity, the treatment should be immediately stopped.
- IN ADDITION:**
 - do not perform the treatment on patients with gingival problems.
 - do not perform the treatment on patients who exhibit an allergy to hydrogen peroxide.
 - patients with dental conditions that could provoke teeth sensitivity or that allow the gel to penetrate inside the tooth (for example: persons with previous tooth sensitivity, receding gums, exposed dentine or roots, teeth with micro-fractures, open cavities, loose dental fillings) – they might be subject to increased or prolonged sensitivity or to temporary pain after the BLANCONE treatment.
 - keep the kit out of the reach of children.

FIRST AID:

IN CASE OF ACCIDENTAL CONTACT WITH SKIN, TISSUES OR BUCCAL MUCOSA: immediately wash and rinse the area thoroughly and repeatedly. Apply a mix of baking soda and water on the irritated area, or a Vitamin E cream. If symptoms persist, immediately seek medical advice.

CONTACT WITH EYES: open eyes and wash with running water for 15 minutes. If burning persists, immediately seek medical advice.


INGESTION: if swallowed, rinse mouth with water and immediately seek medical advice.

INHALING OF POWDER: breathe outdoors. If respiration problems persist, immediately seek medical advice.

TOOTH DISCOLORATION TREATMENT PROTOCOL


A. Active gel preparation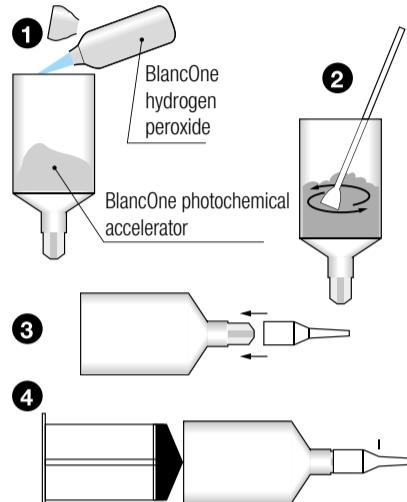

WARNINGS:
Always wear protective gloves and eyewear when handling hydrogen peroxide 39%

1. Before beginning, remove the small cap from the BLANCONE powder mix dispenser syringe.
2. While holding the BLANCONE hydrogen peroxide ampoule, put the entire contents into the syringe containing the powder, keeping it upright.
3. Thoroughly blend the powder with the liquid, making sure that no lumps or residues remain on the sides of the syringe. Mix until a soft viscous, creamy and homogenous gel is obtained (about 20 seconds). When ready, the active gel will be a brilliant orange color.
4. Once the gel is ready, it must be used within one hour from its preparation. After 1 hour the gel will have lost its original effectiveness and is no longer suitable for application, even if stored in the refrigerator.
5. Now remove the cap of the syringe with the gel and insert the white plastic tip, twisting it to make sure it is correctly locked into position.
6. Insert the piston into the syringe with the gel. Place the syringe tip against a piece of gauze and push the piston to eliminate any excess air, concentrating the gel towards the bottom of the syringe.
7. Insert a mouth prop and tongue guard into the patient's mouth along with cylindrical pads above and under the lips to isolate them from the treatment area.

B. Application of the protective gingival gel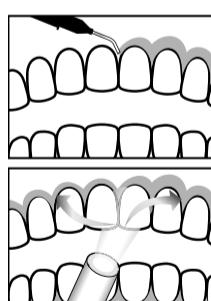


Before beginning the treatment, the tissue around the teeth to be treated must be completely and accurately isolated.

- after inserting the needle on the protective gingival syringe, extrude the protective gel around the gingival edge of the upper teeth to be treated. For added safety, extend the protection by one tooth beyond the targeted treatment zone. The syringe contains enough gel to repeat and reinforce the protection of the gums, if needed and as required.
- Once the protective gel has been applied, start hardening (curing) it by using a normal dental curing light.
- After applying the protective gingival gel on the upper dental arch, repeat the same operation on the lower dental arch.
- Cut off the light beam immediately after the protective gel has hardened.
- Check that all the protective gel has hardened and that the protection has been properly applied.

C. Teeth Discoloration treatment

C1) Active gel application

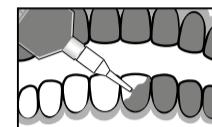1. Apply the active gel on all discolored teeth: gently push the syringe piston to extrude the gel onto the exterior of the teeth. Make certain that the gel layer is uniform and thin: approximately 1 mm. If the layer of gel applied is too thick, use the tip to remove the excess gel until the layer has the proper thickness.
2. Start from the upper dental arch and continue onto the lower arch. During the next cycle, reverse the starting order of the dental arches.

C2) Active gel activation

The biophotonic active gel is activated using a dental curing light available in the office as described in the BLANCONE protocol (see point 4). As long as the gel is orange, it has not been activated. It is recommended not to leave the non-activated gel on the teeth for an extended period of time. Once the gel has been applied on both arches, the light activation procedure should be started immediately, tooth by tooth. Hold the tip of the curing light about 5 mm from the surface of each tooth being treated.

ACTIVATION CONTROL

The time needed to activate the gel will vary depending on the type of light used and its power.


The activation may require anywhere from about 5 to 20 seconds for each tooth.

- The minimum light power required is 1000 mW/cm².
- The peroxide has been activated when the gel turns from orange to a bubbly white color.
- This tells the operator to move the light immediately to activate the gel on the next tooth to be treated, repeating the same process over the entire arch.

ACTIVATION WITH BLANCONE ARCUS LIGHT

The BlancOne Arcus arc light simultaneously activates the gel on both arches, which speeds up the entire procedure.

C3) Gel removal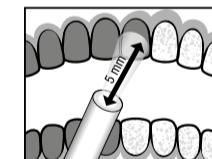


Once the gel on the last tooth has been activated, use the saliva aspirator to remove the activated gel from all teeth on both arches.

C4) C1-C3 Repetition FOUR-CYCLE PROTOCOL

According to the BLANCONE protocol, the gel application, activation and removal operations must be repeated 4 times in a non-stop sequence, as described above.

A fifth and sixth optional cycle can be carried out, in complete safety, if deemed necessary by the operator based on the presence of specific problems.

D. Completing the procedure: removing the protective gingival gel

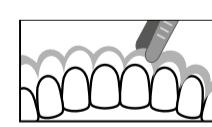After the gel from the fourth or final cycle has been suctioned off, remove the protective gingival gel using appropriate dental tools.

Carefully aspirate and rinse out the mouth with plenty of water. Do not rinse with any mouthwash containing fluoride or chlorhexidine.

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