BLANCONE[®]

WASH

Teeth whitening mouthwash

EN

Collutorio sbiancante

IT

www.blancone.eu/pro

IDS Spa
17100 Savona - Italy - Via San Cristoforo, 28/10
Tel. +39 019 862080 - www.idsdental.it

Made in Greece

Rev. 0 - 11/12/25

Instructions for use

Read all this leaflet carefully before you start using this product. Keep this leaflet, you may need to read it again.

1. What is BlancOne® WASH and when use it

BlancOne® WASH is a mild antiseptic cleanser and teeth whitening mouthwash based on Carbamide Peroxide 5% (Hydrogen Peroxide 1,75%).

While hydrogen peroxide is very unstable and quickly deactivated by oral enzymes, BlancOne® WASH is based on Carbamide Peroxide which gradually splits into Urea and Hydrogen Peroxide ensuring a prolonged activity and oxygen release. In addition, urea is highly effective in degrading plaque biofilm where viruses can bind to.

BlancOne® WASH has a lightly viscous consistency providing a through cleansing action, limiting foaming and is spearmint flavored to allow a comfortable and prolonged rinsing (30-60"), necessary to ensure proper mouth cleansing, protection and micro bioload control.

BlancOne® WASH, used 1-2 times a day, will provide a whiter smile in 2 weeks.

BlancOne® WASH also includes Sodium Hyaluronate to nourish and protect soft tissues and has natural pH for tooth and gums.

BlancOne® WASH is completing and increasing the effectiveness of ORALSAN Chlorhexidine mouthwash providing extra antimicrobial protection and eliminating the possibility of staining the soft tissues of the oral cavity or tooth.

2. How to use

Use 10 mL of BlancOne® WASH for 30 sec, 1 - 2 times per day. Do not dilute with water. Spit out after the use and do not rinse the oral cavity.

3. Possible side effects

If you experience any of the following serious effects STOP using this product immediately: Urticaria, skin rash and itching, allergic reactions.

4. Storage

Keep out of the sight and reach of children. Do not store above 25°C. Store in original package. Do not use BlancOne® WASH after the expiry date stated on the label or carton.

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