PROFESSIONAL

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Rev. I (10 Giugno, 2015)

#### RECOMMENDATION FOR USE

SENSOUT® Professional is a silicone based one component acid protecting desensitizing varnish with Nano-Fluorapatite, Nano-Calciumfluoride.

SENSOUT® Professional is for desensitization in cervical regions and free dentine areas as well as to protect and reinforce demineralized enamel, after dental treatments.

SENSOUT® Professional does not require etching. It builds a protective film , that can be replenished, if necessary. The application is easy and time saving.

Each professional desensitizing treatment SENSOUT includes SENSOUT® toothpaste and SENSOUT® toothbrush for sensitive teeth, to continue the professional treatment of sensitivity at home.

#### Indications

- treatment of hypersensitive cervicals
- desensitizing after scaling, professional tooth
- cleaning application after bleaching procedures

#### Application of SENSOUT® Professional

Clean the sensitive areas and rinse thoroughly. Gently dry in an oil and water free air stream.

PROFESSIONAL

#### Before application vigorous shake the bottle with the SENSOUT® Professional. During shaking listen to the glass bead inside the bottle.

Apply SENSOUT® Professional several times in an even thin layer. Avoid contact of the SENSOUT® Professional with the mucosa.

Let SENSOUT® Professional dry for **1 minute** or dry it gentle in an oil free air stream.

If the patient still reports sensitivity phenomena, repeat the application of SENSOUT® Professional.

#### Important remarks

- Do **not** use SENSOUT® Professional under bridges and other restorative materials.
- The intense smell of SENSOUT® Professional volatilizes during drying within 1 minute.
- SENSOUT® Professional contains ethylacetate. Ethylacetate is highly flammable. Keep away from sources of ignition. Avoid inhalation of ethylacetate vapour. After eye contact rinse out with plenty of water and call an ophthalmologist. After contact with skin, wash with plenty of water and soap.
- Avoid contamination of the SENSOUT® Professional with dirty disposable brushes.
- Impressions with vinyl silicones can be made earliest 4 weeks after the treatment with SENSOUT® Professional.

#### Storage:

Do not store above 25 °C (78 °F)! Do not use after expiration date!

#### Warranty

IDS warrants this product will be free from defects in material and manufacture. IDS makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and IDS's sole obligation shall be repair or replacement of the IDS product.

#### Limitation of Liability

Except where prohibited by law, IDS will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

#### Keep away from children! For dental use only!

Rev. I (June 10, 2015)