BLANCONE  
**Clean&White**  
**Xpower**

## Prophylaxis Powder

CE

MD

**IDS**

Instruction Manual	EN	2
Manuale d'istruzioni	IT	4
Gebrauchsanleitung	DE	6
Manuel d'instructions	FR	8
Manual de instrucciones	ES	10
Manual de instruções	PT	12
Gebruikshandleiding	NL	14
Brugsanvisning	DA	16
Käyttöopas	FI	18
Bruksanvisning	NO	20
Bruksanvisning	SV	22
Instrukcja obsługi	PL	24
Návod	CZ	26
Návod	SK	28
Használati utasítás	HU	30
Manual de instrucțiuni	RO	32
Руководство по эксплуатации	RU	34
Talimat El kitabı	TR	36
دليل التعليمات	AR	38

## Prophylaxis Powder

## 1. Specification

Model	Main Ingredient	Volume	Description
SBC65	NaHCO <sub>3</sub> , Na <sub>2</sub> CO <sub>3</sub> and SiO <sub>2</sub>	300g/bottle	FAST – Supra gingival
GLC65	C <sub>2</sub> H <sub>10</sub> NO <sub>4</sub> and SiO <sub>2</sub>	200g/bottle	
GLC25	C <sub>2</sub> H <sub>10</sub> NO <sub>4</sub> and SiO <sub>2</sub>	120g/bottle	MILD – Supra & Sub gingival

## 2. Brief introduction

This powder is used together with professional dental powder jet scaler to remove biofilm and dental plaque. It should be used by professionally trained and qualified doctors or dentists.

## 3. Indication of use

Model	SBC65	GLC65	GLC25
Particle Size	65µm	65µm	25µm
Flavor Composition	sodium bicarbonate sodium carbonate silicon dioxide	glycine silicon dioxide	glycine silicon dioxide
Packing Specifications	300g	200g	120g
pH	8.1	6	6
Implant	✗	✗	✓
Orthodontia Brackets and appliances	✗	✗	✓
Restorations	Hybrid Composite	✗	✓
	Glass Ionomer	✗	✓
Soft tissues	Gingiva	✗	✓
	Tongue and palate	✗	✓
Stains and early calculus	Enamel	✓	✓
	Enamel	✓	✓
	Enamel	✓	✓
Teeth	Enamel whitespots	✗	✗
	Pits and fissures	✗	✗
	Dentine	✗	✗

#### 4. How to use

This powder is used together with professional dental powder jet scaler to remove biofilm and dental plaque. It should be used by professionally trained and qualified doctors or dentists.

- 4.1 Shake the bottle for several seconds before open the bottle.
- 4.2 Open the bottle and fill the powder into the sand blasting equipment, then tighten the cap of the bottle.
- 4.3 Turn on the sand blasting equipment, aim the nozzle of the sand blasting handpiece at the position that need to be treated.

#### 5. Contraindications

- 5.1 Under no circumstances should the sand powder and sand blasting equipment be used to treat patients with chronic bronchitis or asthma. The sprayed water and sand powder mixture may cause breathing difficulties for patients.
- 5.2 Do not aim the nozzle of the sandblasting handpiece directly at the dental filling, crown, denture frame, etc. It may damage the restoration.
- 5.3 SBC65 powder should not be used to treat patients on a low-salt diet.

#### 6. Storage and Shelf Life

- 6.1 Sandblasting powder shall not be placed near acidic liquid (gas) and heat sources.
- 6.2 Store in a dry place at room temperature away from light and place it in a high temperature and high humidity environment for a long time will shorten the validity period of the product; if you find abnormal phenomena such as discoloration and flatulence of the sandblasting powder, stop using it.

#### 7. Precautions

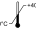
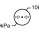
- 7.1 Do not pour the sandblasting powder that is not used up into the sand bottle after use.
- 7.2 After opening the bottle of sandblasting powder, please close it tightly to avoid the sandblasting powder being affected by moisture.
- 7.3 When the sandblasting machine is not in use for a long time, please clean up the remaining sandblasting powder in the sandblasting bottle of the machine to avoid caking and causing the equipment to be blocked.

#### 8. Packing and shipping

- 8.1 Mixing with acidic substances and dangerous goods should be avoided during transportation.
- 8.2 The sun and rain should be avoided during transportation.

#### 9. Environmental protection

#### 10. Symbol Description

	Trademark		Shake the bottle before using
	Caution		CE Mark
	Date of manufacture		Manufacturer
	Keep dry		Humidity limitation: 10%-93%
	Temperature limit: -20°C - +40°C		Atmospheric pressure limitation: 70kPa-106kPa
	Fragile, handle with care		Recovery
	Refer instructions to manual/booklet		Medical device

#### 11. Other usage suggestions and instructions

- 11.1 It is recommended to apply an appropriate amount of Vaseline to the patient's lips before treatment.
- 11.2 Patients and users must wear protective masks or goggles during treatment.
- 11.3 The user must wear a mask during treatment.
- 11.4 During the treatment, please use the high-speed emptying device (strong suction) on the comprehensive dental treatment machine to absorb the sprayed water and sand mixture.

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Guilin Refine Medical Instrument Co., Ltd.
MedNet EC-REP C IIb GmbH
MediMap Ltd