### Snap -n- Go™

Two-tone plaque disclosing swabs | Tamponi rivelatori di placca a due toni | Zweifarbige Plaque-Indikatorstäbchen | Tampons révélateurs de plaque à deux teintes | Hisopos reveladores de placa en dos tonos | Cotonetes para identificar placa de duas cores | Tweekleurige plakverklipper wattenstaafjes | Tofarvet plakvatpind | Plakkiväripuikko | Tofarvede vattpinner til plakk | Bomulspinnar för plackdetektion med två färger | Wymazówki dwutonowe do wykrywania płytki nazębnej | Dvoubarvné tampóny pro detekci plaku | Dvojtónové tampóny na detekciu zubného povlaku | Kéttonúsú lepedékfeltáró tamponok | Tamponos detectores de placă bacteriană în două tonuri de culoare | Двуцветный индикатор зубного налета | Iki tonu dişi tespit için sürüntü çubukları

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
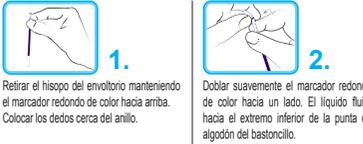
### GB Snap-n-go™ two-tone plaque disclosing swabs

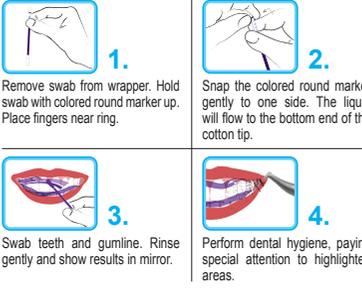Each Clean&White Reveal Snap -n- Go Swab contains an indicating dye that attaches to plaque on teeth and gums. This is a helpful tool to improve professional dental hygiene. guiding biofilm disclosure and removal. NEAT and EASY for the dental professional to apply. A perfect adjunct for oral hygiene instruction in the dental office. TO USE: Before starting professional dental hygiene, apply Clean&White Reveal plaque disclosing to the teeth by dabbing swab along the teeth and gums. Allow to remain in place for 10 seconds. Gently rinse with water. Examine teeth for areas that appear blue and reddish-pink. Blue areas indicate where old plaque is present and reddish-pink areas indicate the presence of new plaque. Use this to guide plaque removal with scaler tip and air prophylax powders. CAUTION: Children 6 years and younger should only use under direct adult supervision. For dental use only. Do not swallow. Reveal contains red and blue dyes, which will stain clothing and other materials if allowed to come in contact with them.

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